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Preface

Thank you very much and congratulations on your choice.

We are glad you choose a product by hofmann gmbh.

By means of infrared technology this thermometer determines the body temperature in a matter of a few seconds by measuring the warmth send out from the ear drums or the forehead.

For each thermometer the correct measuring technique is essential supposition to achieve exact measurements. Please read this instruction manual thoroughly prior to using your sanowell® Scaneo.

The sanowell® Scaneo thermometer has been developed under strict precision and safety requirements for measurement of temperature in the ear and at the forehead.


It is equipped as follows:

- Three units in a unique design. It can measure temperature in the ear as well as at the forehead and can also determine the temperature of the environment, which means that you will only need one device for these three functions.
- Display of time.
- Instant measurement: The result can be read within seconds.
- The device does not need protective probe covers, as the probe (not the entire unit) is waterproof and therefore easily disinfected.
- Exact high speed measuring.
- Automatic change from forehead measuring mode to ear measuring mode by simply removing the tip cover. It is very comfortable and simple to use.


Date of edition: April 01, 2003

sanowell® Scaneo thermometer

Front view: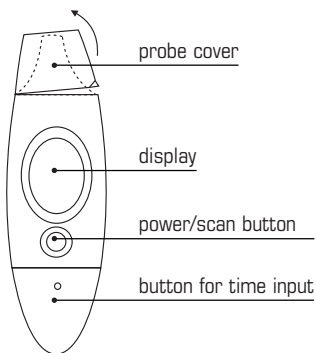

Rear view:

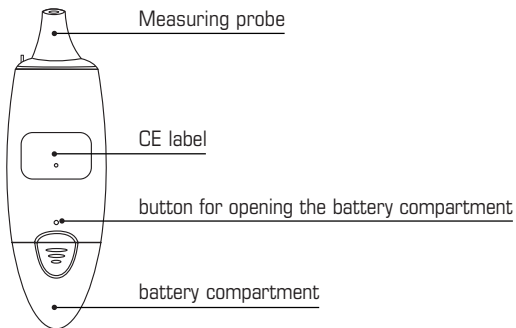Explanation of symbols

ongoing temperature measurement

mode of temperature determination in the ear

mode of temperature determination at the forehead

Centigrade scale

Time

low battery

saving function for determined temperature

Caution


- The measuring probe is the most delicate part of your thermometer. It has to always be clean and intact to warrant exact measurements.
- The body of the thermometer is not waterproof. Do not immerse into liquids.
- The thermometer may not be subjected to extreme temperatures for a longer period of time like for example in a vehicle directly exposed to sunshine.
- The thermometer may not be subjected to falls, as it might break and the warranty becomes invalid. The probe should not be touched directly.
- The thermometer may only be disassembled by professionals.
- This device is meant only for household use and does at no time substitute medical treatment.
- The ear should be clean and free huge amounts of ear wax.
- It is possible that temperature measured in the right ear differs from the one in the left minimally.
- Incorrect measurements are possible, if measurement is taken in an ear the has been laid on for a longer period of time. In this case, please wait a few minutes before measuring.

Time respectively environmental temperature display

When using the thermometer for the first time, or after a battery exchange, you will first have to set the time. After that the environmental temperature measuring mode is activated.


Setting time

Open the battery compartment by pressing a pointed article on the button on the battery compartment and pull away the compartment cover. After that please press the button for time input until the display for hours blinks. The hours are set by pressing the power/scan button repeatedly (until the desired setting for hours is achieved). By pressing the time input button once you switch from setting the hours to setting the minutes. When the display for minutes blinks proceed exactly as above described for setting the hours. Once time is set, please press the time input button one time to finish.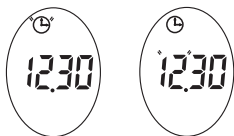

Modes of display

There are two different options for display. One shows time and environmental temperature (the display switches between time and environmental temperature; see picture), the other informs you about body temperature (body temperature is displayed; see picture).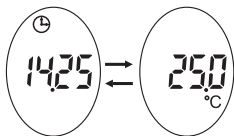

Measuring body temperature


In order to switch from measuring at the forehead to measuring in the ear simply remove the plastic cover. To return to forehead measuring mode reapply the cover.

Caution

It is recommended to measure temperature at the forehead just for control, as the skin temperature is prone to outer influence and the result may be false.


Measuring of body temperature in the ear

For measuring temperature in the ear remove the tip cover and press the power/scan button. In case of display of time and environmental temperature press the power/scan button again. The latest temperature determination is shown (see picture). When the symbol for ear measurement is displayed the device is ready for use. Now press the power/scan button again and when the symbol for ongoing measurement is displayed insert the probe into the ear. After a few seconds the body temperature is displayed. Hold the device steady while measuring. After the measuring the device automatically returns to the time/environmental temperature mode.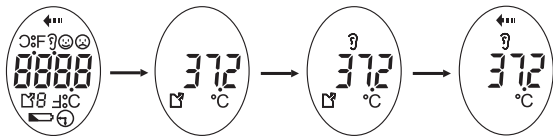

Measuring temperature at the forehead

To determine body temperature at the forehead make sure that the probe cover is in place. In case of display of time or environmental temperature press the power/scan button. The unit beeps once and the latest temperature determination is displayed. Now press the power/scan button again and the symbol for measurement at the forehead will be shown. To start measuring hold the device to the forehead and at the same time press the power/scan button. After completion of measuring the device will beep once again and the body temperature is displayed. A few seconds later the device shuts off and returns to the time/environmental temperature mode.

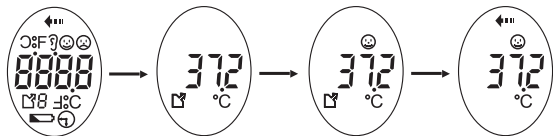Optimum measuring conditions

The thermometer is equipped with a function to optimise the conditions during measuring procedure. In case the device is not ready for measuring yet, either the ear or the forehead measuring mode symbol illuminates (see picture) and the unit expels short, quick, consecutive signal tones. (Press the power/scan button to stop temperature measuring). Do not continue with the measuring procedure until the symbol is unlit or as long as the signal tone sounds.

Battery control display


The device is delivered with a lithium cell (3 Volt, CR 2032). In case of low battery the according symbol is shown in the lower part of the display. It is recommended to exchange the battery as soon as possible, although, if necessary, it can still be used (see picture). If the battery is very low, additionally to the blinking battery symbol the „Lo“-symbol is displayed and a signal tone sounds (see picture). Now it is not possible anymore to use the thermometer unless changing of the battery. In this case press any button to turn off the unit.

Changing the battery

The present thermometer is equipped with a safety lock to keep the battery away from children. Press down the button with a pointed article and then remove the battery.

You need a 3 V CR 2032 lithium battery for your device. When inserting the battery the negative pole has to be turned to the bottom.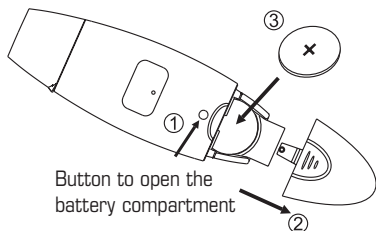

Advice for temperature measuring

- The thermometer is equipped with a waterproof probe, which excludes the necessity of using a protective probe cover.
- Keep the probe clean. Avoid accumulation of ear wax, tear drops or oils, as they might influence the results of the infrared measuring.
- Do not determine your temperature within half an hour after a meal, physical exercise or a bath.
- Please leave an interval of one minute between consecutive measurements. When measuring temperature often or within a short period of time, results may vary slightly. In this case take the mean value for orientation.
- Human body temperature changes within a naturally given scale. There is no absolutely set value for standard body temperature.
- Please take notes about your body temperature on regular days to better determine your febrile values.
- In the event that environmental temperature conditions are outside the allowance of 16°C - 40°C the display will show „Err“ when measuring the environmental temperature.

Care and cleaning

The probe is the most delicate part of the device. It has to be kept clean and intact in order to warrant exact measurements.

The probe is waterproof and may be cleaned by either immersing into alcohol or cleaning with a cotton pad impregnated with alcohol. The body and the display of the thermometer are not waterproof. Do not immerse into liquids, but clean with a soft clean cloth. Do not use aggressive cleaning agents. Store the thermometer at a dustfree, dry location and keep away from direct sunlight. The temperature at the storage location should be in the range of 10°C to 40 °C.

Technical specifications

Displayed temperature

range: 10 - 50 °C

Measuring accuracy: At less than 36,0°C or more than 39,0 °C
+/- 0,3 °C. At 36,0 °C to 39,0°C +/- 0,2 °C

Display resolution: 0,1 °C

Conditions of use: 16,0 °C to 40,0 °C at no more than 95% relative
humidity

Conditions of storage: -20,0 °C to 50,0 °C at no more than 95 % relative
humidity

Battery: One 3 V CR 2032 lithium battery

Battery life span: about 500 measurements

This device is manufactured according to the EC-guidelines 93/42/EWG

Calibration

The thermometer has been calibrated at manufacture. When used as prescribed in this instruction manual no additional calibration is necessary.

Explanation of signs on the CE label

Application component of type BF. Application components are not suitable to be used for direct application at the heart.

Advice on recycling

The packaging ought to be disposed of at the garbage recycling container, the metal parts of the device at the used metal recycling station. Plastic pieces, electronic parts, and conducting boards are to be disposed of as electronic waste. Batteries go to the special garbage collection. Please render them to your Recycling plant and do not throw them into your regular household waste. For further information about recycling and disposal of special waste turn to your local recycling plant.

Warranty

There is a 2 year guarantee for our products, applicable from date of purchase.

Within the prescribed warranty time we will remove all defects of the device, that are based on faulty material or manufacture. Removal of defect occurs at our choice by either repair or replacement of parts or the entire device.

Exempt from warranty are: Defects caused by inappropriate use (f. ex. connection to inappropriate electrical current or voltage, connection to inappropriate sources of electricity, breakage by fall and such). Normal wear and tear that only minimally affects the operational utility of the unit is not subject to the warranty.

Warranty subsides if unauthorized intervention or repair is undertaken, or if non-original parts are used.

Warranty is effective only, if date of purchase is confirmed on the warranty card by stamp and signature of the dealership.

In case of warranty claims, please return your device completely with the warranty card to one of our authorized dealerships or send both to the nearest customer service.

No other claims are valid under the provisions of this warranty, unless our liability is based on legal statute. Claims against the dealer based on the provision of the purchase agreement do not affect this warranty.

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